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C001-1 30mLx1
C001-2 60mLx1
C001-3 100mLx1

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C001-1 30mLx1
C001-2 60mLx1
C001-3 100mLx1

Explanation of Symbols

	Sterilized using aseptic processing techniques
	Batch code
	Date of manufacture
	Use-by date
	Temperature limitation
	Keep away from sunlight

Precautions and Warnings

1. Before using this product, users should read and understand the Operation and precautions.
2. This product cannot be used for injection.
3. This product is sterilized by filtration and is not suitable for re-sterilization.
4. Do not use this product if:
The product package is damaged or the seal has been destroyed.
 - The product exceeds the shelf life.
 - The product has become turbid or granular.
5. Discard unused products after equilibration.
6. This product is suitable for single use. After opening the bottle, discard the remaining unused product.
7. In order to avoid contamination, the media should be used only with aseptic technique.
8. This product can be used with SAGE, LifeGlobal, and Vitrolife products of mainstream companies in the market.

SAGE:

Quinn's Advantage™ Medium with HEPES(ART-1023)

Quinn's Advantage[®] Fertilization Medium(ART-1020)
Quinn's Advantage[®] Cleavage Medium(ART-1026)
Quinn's Advantage[®] Blastocyst Medium(ART-1029)
LifeGlobal:
global* total* lp with HEPES(H5TH-050)
global* total* for Fertilization(H5TF-050)
global* total* lp(H5GT-030)
Vitrolife: G-MOPS™ plus (10130) 、G-IVF™ plus (10136) 、G-1™ plus (10128) and G-2™ plus (10132)
Mouse embryo assay (MEA) - 1-Cell MEA> 80%, pregnancy rate is 100%, Other manufacturers' products have not been tested.

10. Caution: Federal law restricts this device to sale by or on the order of a physician.

Intended use

Oil for covering of medium or micro-manipulation procedures during in vitro fertilization

Composition

Paraffin oil

Storage and Transportation

Store dark at 2°C ~ 8°C for 2 months.

Store dark at 2°C ~ 8°C during transportation.

Performance

1. Endotoxin: <0.1EU/mL.
2. Sterility testing: Negative.
3. 1-Cell Mouse embryo assay(%): ≥80%.

Operation

1. For use after equilibration at 37°C and 5 % CO₂ or 6 % CO₂ at least 16h
2. Remove Oil from vial using a sterile, non-toxic pipette. Do not pour.
3. Using a sterile, non-toxic pipette, add culture medium to a vessel labeled with patient ID and cover entirely with Oil.